
NOVIDES BT-BINX 20220401 V6

How to use **BTNX** COVID-19 rapid antigen tests



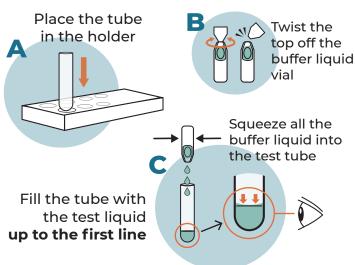

2 COLLECTION



Compress the nostril with your finger as you turn the nasal swab

Insert nasal swab up to **0.5 inch (1.3 cm)** from edge of nostril. Roll **5 times** in **each nostril** (right & left)

3

PREPARATION

Insert the swab in the tube liquid and rotate it 10 times



Place the tube and swab in the holder and leave it there for **2 minutes**



Remove the swab by pinching the tube

Close the tube with the stopper cap



TESTING


Place **3 drops** in the rapid response cartridge and wait **15 minutes**



(Do not touch it during the waiting time)

POSITIVE

With a positive result, the "T" line

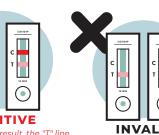

Redo the nasal swab with a new kit.

If the test result is positive: Please inform your PSO.

may appear red or pink (as shown).

Here is an example of a positive test, negative and an invalid test

